

INTRODUCING THE ARGYLE™ SAFETY FISTULA CANNULA

Finally, there's a flexible choice for you and your dialysis patients. Leveraging the progress made in hemodialysis treatment and technologies in Japan during the past 30 years, Medtronic offers a flexible fistula cannula. With an improved vascular access design meant to enhance clinician safety and patient comfort, the **Argyle™ fistula cannula demonstrates that flexibility in dialysis can be yours.**

87.5%¹

Patients who preferred the Argyle™ fistula cannula to a steel needle

"The Argyle cannula needle has changed my experience in dialysis... I wish they'd had these things around when I first started dialysis!"

WAYNE GRIGSBY
25-year dialysis patient
Victoria, Australia



The Argyle™ fistula cannula has two parts – a sharp metal **needle** with an integrated safety mechanism and a blunt **cannula**. Once the metal needle is removed, the safety mechanism is activated covering the sharp tip thus leaving only the flexible cannula in-situ for hemodialysis. This feature is designed to prevent needle-stick injuries.

Medtronic
Further, Together

Vascular Access

- The cannula is designed to accommodate varying vessel depths, including deep access.
- The cannula is DEHP- and latex-free.
- The plastic material is duller and more flexible than a steel needle.
- The plastic cannula flexes to accommodate tortuous vessels.
- Studied in comparison to a steel needle, the plastic cannula reduced the likelihood of hematoma.²
- In simulation, the plastic cannula did not infiltrate the AVF/AVG wall as compared to a steel needle, which damaged the artificial vessel.³

Safety

- An integrated safety needle-stick prevention mechanism covers the needle tip when the puncture needle is removed.
- There is no need for the clinician to activate the safety mechanism.
- The Argyle™ safety fistula cannula is designed to prevent needle-stick injuries, which aligns with EU recommendations to protect healthcare workers and patients.⁴

Patient Comfort

- The plastic cannula flexes to accommodate patient movement during the dialysis.
- In a survey of 40 dialysis patients, 87.5% preferred the Argyle™ fistula cannula over a steel needle to access their fistula or graft.¹

ITEM CODE	CANNULA		COLOR
	O.D. (mm)	Length (mm)	
1538VSVS	1.8 (15G)	38	Blue-Gray
1530VSVS	1.8 (15G)	30	Blue-Gray
1638VSVS	1.6 (16G)	38	White
1630VSVS	1.6 (16G)	30	White
1738VSVS	1.4 (17G)	38	Red-Violet
1730VSVS	1.4 (17G)	30	Red-Violet

The order unit is a case containing 50 eaches.



1. Data on file, Argyle™ Fistula Cannula Patient Survey, 2015.

2. Letachowicz K, Kusztal M, Golebiowski T, et al. Use of Plastic Needles for Early Arteriovenous Fistula Cannulation. Blood Purification. 2015;40:155–159.

3. Data on file, Argyle™ Fistula Cannula Marketing Claims Support Study Test Report, 2016.

4. Directive 2010/32/EU of the European Council on implementing the Framework Agreement on prevention from sharp injuries in the hospital and healthcare sector concluded by HOSPEEM and EPSU, 2010. O.J. L 134/66.

Photo credit: iStock

Medtronic

IMPORTANT: Please refer to the package insert for complete instructions, contraindications, warnings and precautions.

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.

™* Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

EMEA-RC-2000024 - 20-weu-argyle-fistula-cannula-sell-sheet-4811087

medtronic.com/covidien/uk