



EMODIAL

Medtronic

exclusive distributor in Canada

SOVAN[®]

Fistula Dressing



EMODIAL

Via Finati, 4/A - Ferrara (ITALY)
Tel. (+39) 0532 978198 - Fax (+39) 0532 978414
www.emodial.it - info@emodial.it



REV. 11/2018



Description:

Highly absorbent pad set on hypoallergenic breathable adhesive.

Indications:

Compressive haemostatic action, specific for post-dialysis arterovenous access care to be used after the removal of the fistula needles.
Also suitable for patients who require intravenous treatment and blood transfusions.

Purpose:

Designed to stop bleeding and to reduce the haemorrhagic risk, thanks to the compression on the needles holes, and to work as a barrier against the risk of infection.
The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of bleeding**.
The adhesive helps to keeps the pad in position.



Directions for use

- 1 Remove protective paper and fix the first half of the dressing to the skin.
- 2 Put the pad over the needle site, press and remove needle.
- 3 Remove the second half of the paper and fix tightly the other half of the dressing to the skin. Hold in place until haemostasis is achieved.



REFERENCES Data and technical specifications can be changed without notice

CODE	NW DRESSING cm	PAD cm	Pcs/ Pack	Packs/ Box
S0079-1	10 x 5	ø 2.6	1	700

Classification:
Biocompatible:
Sterilization:
Expiration:



Class I medical device
according to UNI EN ISO 10993 standards
Gamma rays
5 years
Single-use