



EMODIAL

Medtronic

exclusive distributor in Canada

BETAFIX[®]

Fistula Bandage



EMODIAL

Via Finati, 4/A - Ferrara (ITALY)
Tel. (+39) 0532 978198 - Fax (+39) 0532 978414
www.emodial.it - info@emodial.it



REV. 11/2018

Fistula Bandage

Description:

Elastic fistula bandage provided with a two layer highly absorbent pad and adhesive for a better tightness.

Indications:

Compressive haemostatic action, specific for post-dialysis arterovenous access care to be used after the removal of the fistula needles.

Also suitable for patients who require intravenous treatment and blood transfusions.

Purpose:

Designed to stop bleeding and to reduce the haemorrhagic risk, thanks to the compression on the needles holes, and to work as a barrier against the risk of infection.

The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of bleeding**. The adhesive helps to keep the pad in position.

The cotton bandage avoids the sensitization due to the adhesive of plasters.



Directions for use

- 1 Unroll the first part of the bandage with the pad.



- 2 Put the pad over the needle site.



- 3 Tighten bandage sufficiently to stop bleeding.



- 4 Remove the protective film and fix the bandage with the adhesive.



REFERENCES Data and technical specifications can be changed without notice

CODE	Bandage LENGHT cm	Bandage WIDTH cm	PAD cm	Pcs Pack	Packs/ Box
S0029-1	50 ± 10%	6	ø 2.6	1	500

Classification:
Biocompatible:
Sterilization:
Expiration:



Class I medical device
according to UNI EN ISO 10993 standards
Gamma rays
5 years
Single-use